

# Instructions for Installing DXAnalyzer Professional 1.0 Demo!

Please follow these steps for updating DXAnalyzer Professional software Demo Version.


1. Click on “*DXAnalyze.exe*” link in [www.dxanalyzer.com](http://www.dxanalyzer.com) download section.
2. On “*File Download*” window, click “*Open*” to start downloading. Setup will automatically start after the file gets downloaded onto your machine. Click “*OK*” on “*WinZip Self-Extractor*” window to continue setup.
3. On the “*Welcome*” screen click “*Next*”.




4. Read “*Software License Agreement*” and click “*Yes*”




5. On “*User Information*” screen, Enter your Name, Company and the following serial no: **DXAP10-01-00-2004-12345678** and click “*Next*”

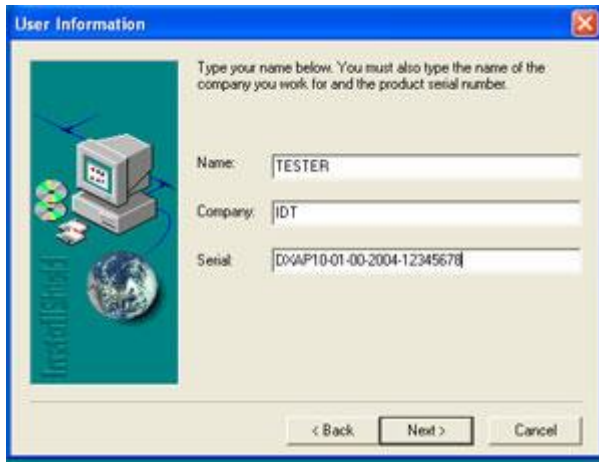


6. On “Set Up” window click “Next”

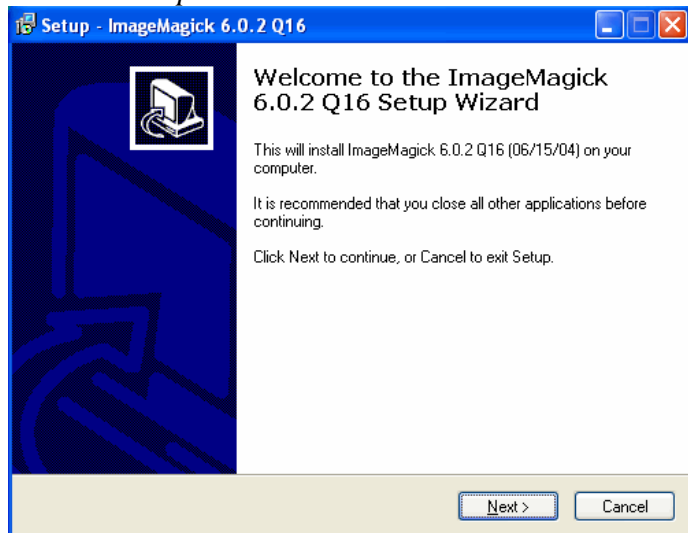


7. Accept the agreement and click “Next”

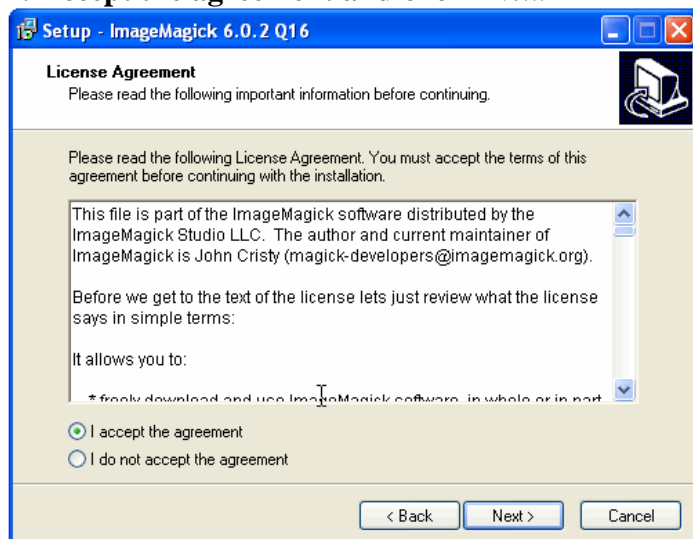


8. Proceed by clicking “Next” until you see “Finish” window and click “Finish”

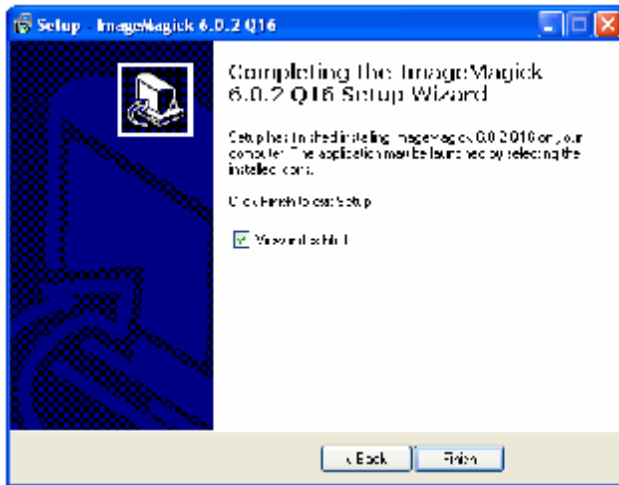

9. On the “Warning” screen click “Yes” to continue

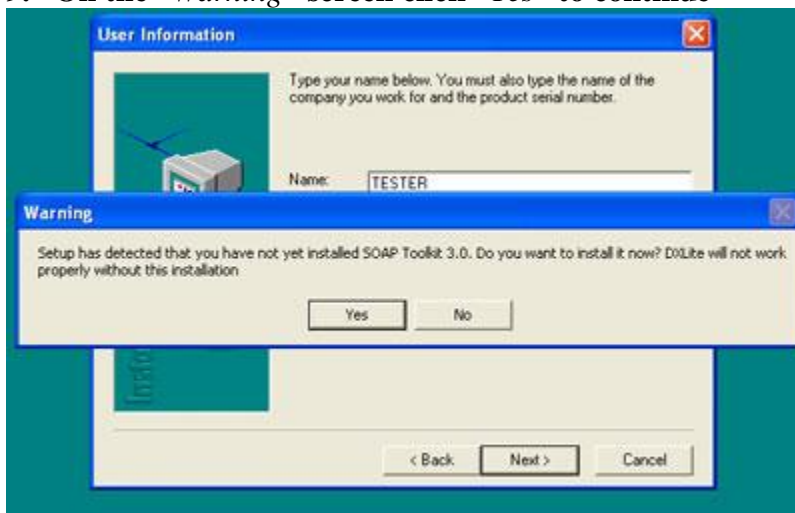

10. Click “Yes” to install SOAP toolkit for internet updates

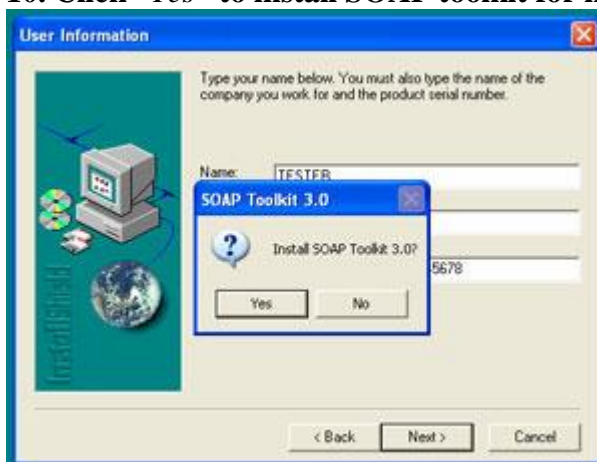


11. Click “Next” to continue installing SOAP toolkit 3.0 setup wizard



12. Read the “*License Agreement*” and select the option of acceptance then click “*Next*”



13. Enter “*Name*” and “*Organization*” and click “*Next*”

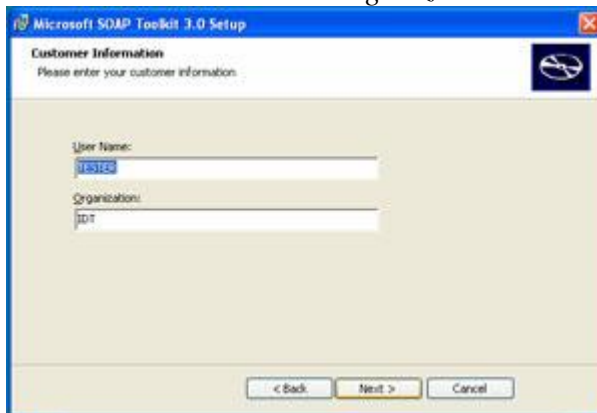


14. On the “*Custom Setup*” screen, Click Install




15. Click *“Finish”* to exit the setup wizard.




16. On *“Setup Type”* screen, click *“Next”*

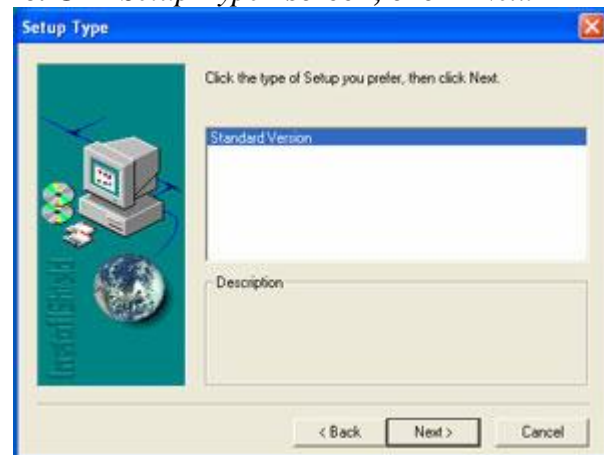


17. On *“Select Program Folder”* screen, click *“Next”*

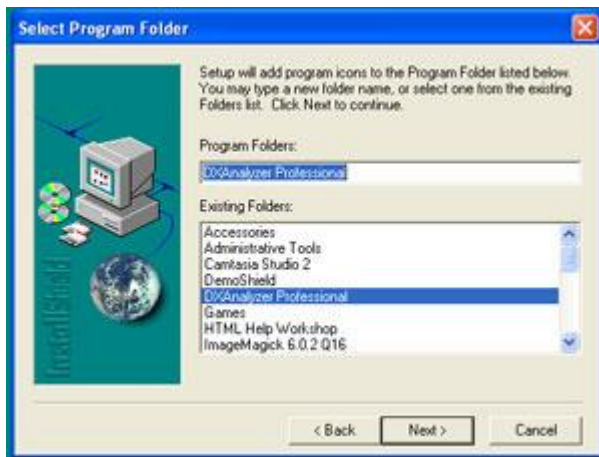


18. Click “Yes” to create a desktop shortcut.

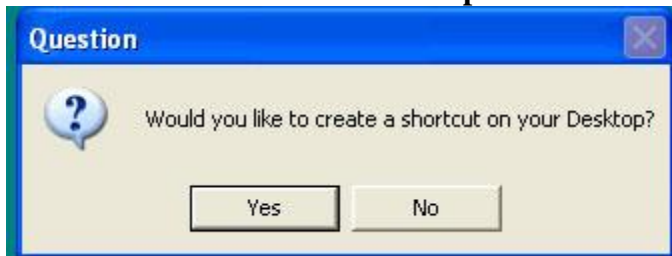

19. Click “Finish” on “Setup Complete” screen

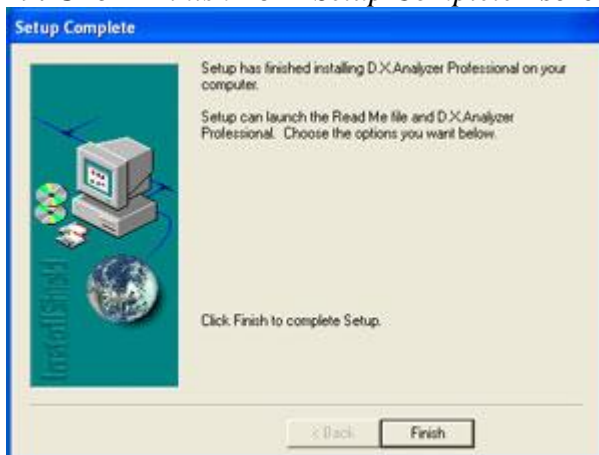

20. Click “Ok” on “Information” screen

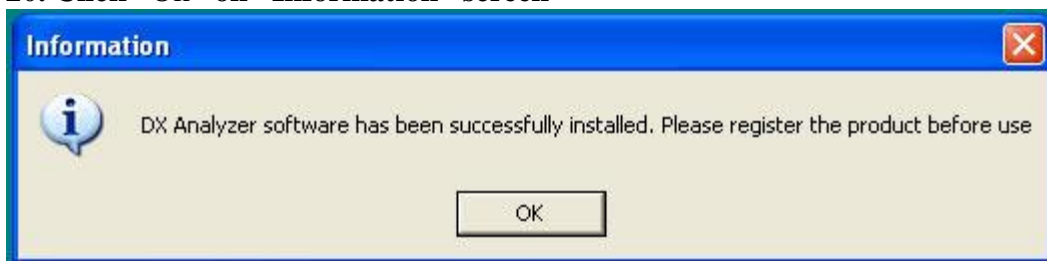

**Now! You are ready to run DXAnalyzer Professional Software!**

*If you have any questions or concerns please contact  
International Diagnostic Technologies at 888-926-2774  
Or  
Email at [support@dxanalyzer.com](mailto:support@dxanalyzer.com).*